

Manus and Gauntlet application guide

How to change Actimove® Manus / Gauntlet to fit the desired hand

- Open Actimove Manus / Gauntlet and place it on a flat surface in front of you
- The hook and loop for the thumb should be pointing to the top right hand, the wrist hook and loops to the left
- There are two openings on the inside towards the lower end of the Actimove Manus / Gauntlet, which are visible when you open the product. One is the lower opening, one the upper one



Manus

Gauntlet

NOTE:

- Insert the stay in the **lower opening** for **left hand** assembly
- Insert the stay in the **upper opening** for **right hand** assembly

Manus



To change to **right hand** (when assembled for the left hand)

- Remove the stay from the lower opening of the Actimove Manus / Gauntlet
- Turn the stay sideways by 180°, so that the top end curves downwards
- Insert into the upper opening, curved end first
- Then apply as normal on the right hand

Gauntlet



To change to **left hand** (when assembled for the right hand)

- Remove the stay from the upper opening of the Actimove Manus / Gauntlet
- Turn the stay sideways by 180°, so that the top end curves upwards
- Insert into the lower opening, curved end first
- Then apply as normal on the left hand

